



# Product Application Optimal Positions

**NO-HEMACLEAR® ZONE  
ARM** ❌  
Ulnar Nerve, Radial Nerve,  
Elbow, Forearm Muscles


**NO-HEMACLEAR® ZONE  
LEG** ❌  
Peroneal Nerve, Knee,  
Calf Muscles -  
Soleus/Gastrocnemius

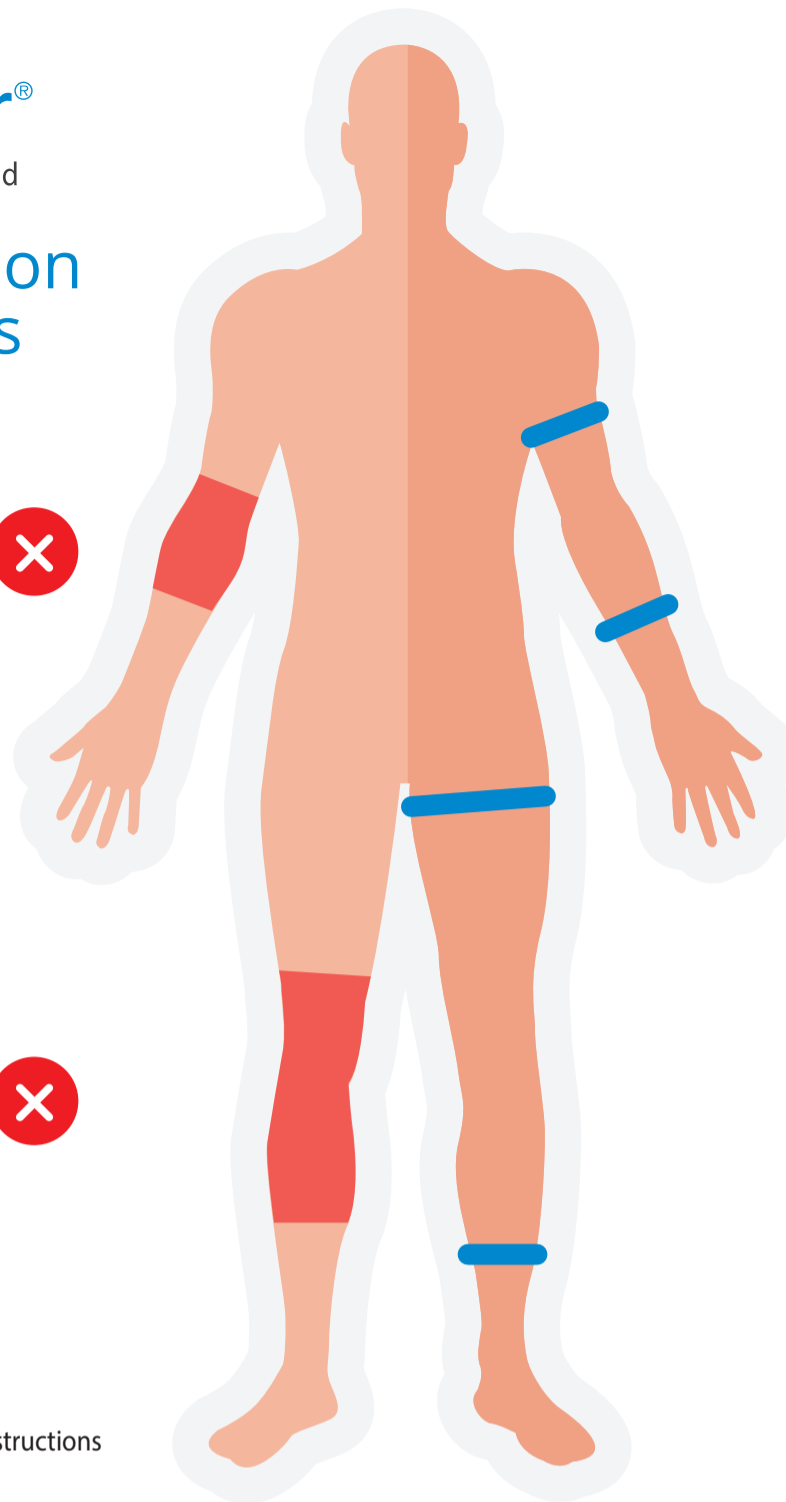


✓ **UPPER ARM**  
Bottom of Deltoid Muscle  
**Medium Yellow** or **Large Orange**

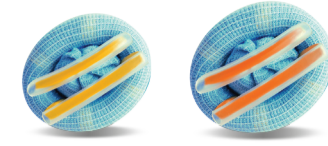

✓ **FOREARM**  
10 cm Above the Wrist  
Distal to Forearm Muscles  
**Model F™**

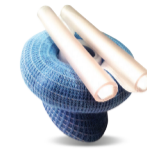

✓ **THIGH**  
As Close to the Groin as Possible  
**Extra-Large Black & White**  
or **Large Orange** or **Large Brown**

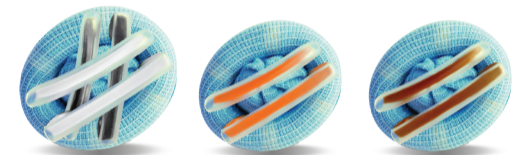

✓ **LOWER LEG**  
10-15 cm Above the Lateral Malleolus  
**Model A™** or **Large Orange**



See the HemaClear® User Guide for detailed instructions

HemaClear® <b>MEDIUM YELLOW</b> <190 mm Hg PRH-040-YE-01A		Distance from Fingers / Toes (cm)					
		20-30	31-40	41-50	51-60	61-70	71-80
LIMB CIRCUMFERENCE (cm)	24-26	231	224	216	208	199	190
	27-28	233	228	222	215	206	196
	29-30	251	242	232	222	214	202
	31-32	270	260	248	236	229	214
	33-34	283	274	265	249	244	226
	35-36	325	304	288	263	259	238
	37-38	NPD	333	314	288	274	249
	39-40	NPD	368	344	339	311	261

HemaClear® <b>MEDIUM RED</b> <160 mm Hg PRH-040-RE-01A		Distance from Fingers / Toes (cm)					
		20-30	31-40	41-50	51-60	61-70	71-80
LIMB CIRCUMFERENCE (cm)	24-26	221	218	212	203	194	179
	27-28	230	225	218	209	198	185
	29-30	238	235	227	217	208	191
	31-32	258	252	242	232	221	204
	33-34	265	263	257	246	235	217
	35-36	300	289	277	261	248	229
	37-38	NPD	332	301	283	262	242
	39-40	NPD	361	327	322	305	254

HemaClear® <b>LARGE BROWN</b> <190 mm Hg PRH-060-BR-01A		Distance from Fingers / Toes (cm)						
		40-50	51-60	61-70	71-80	81-90	91-100	101-110
LIMB CIRCUMFERENCE (cm)	30-32	310	301	296	281	276	264	257
	33-36	362	355	346	337	332	320	311
	37-40	387	379	370	363	354	344	337
	41-44	NPD	429	407	389	376	367	363
	45-48	NPD	NPD	436	412	395	389	384
	49-52	NPD	NPD	NPD	425	409	406	400
53-55	NPD	NPD	NPD	432	422	418	413	


HemaClear® <b>LARGE ORANGE</b> <160 mm Hg PRH-060-OR-01A		Distance from Fingers / Toes (cm)						
		40-50	51-60	61-70	71-80	81-90	91-100	101-110
LIMB CIRCUMFERENCE (cm)	30-32	281	276	264	257	250	236	229
	33-36	341	332	320	311	306	285	281
	37-40	365	355	344	337	326	307	302
	41-44	NPD	423	390	363	346	328	324
	45-48	NPD	NPD	413	372	352	335	324
	49-52	NPD	NPD	NPD	387	357	341	324
53-55	NPD	NPD	NPD	415	365	349	346	

HemaClear® <b>EXTRA LARGE BLACK &amp; WHITE</b> <160 mm Hg PRH-090-BW-01A		Distance from Toes (cm)					
		60-70	71-80	81-90	91-100	101-110	111-120
LIMB CIRCUMFERENCE (cm)	50-54	300	296	292	289	285	281
	55-60	315	311	307	304	300	296
	61-66	327	323	320	316	313	310
	67-72	337	333	330	326	323	320
	73-78	345	342	338	335	331	328
	79-85	352	349	346	342	339	336

HemaClear® <b>MODEL F™ (Forearm)</b> <160 mm Hg PRH-035-FA-01A		Distance from Fingers (cm)		
		25-30	31-35	36-40
LIMB CIRCUMFERENCE (cm)	14-18	204	197	192
	19-22	236	231	227
	23-26	257	254	247
	27-30	276	270	264
	31-34	295	286	281


HemaClear® <b>MODEL A™ (Foot &amp; Ankle)</b> <160 mm Hg PRH-032-MA-01A		Distance from Toes (cm)				
		30-34	35-38	39-42	43-46	47-50
LIMB CIRCUMFERENCE (cm)	22-24	176	167	161	NPD	NPD
	25-26	184	177	170	164	NPD
	27-28	191	184	177	170	165
	29-30	NPD	190	183	177	171
	31-32	NPD	NPD	188	182	176

# Product Application Optimal Positions PEDIATRICS

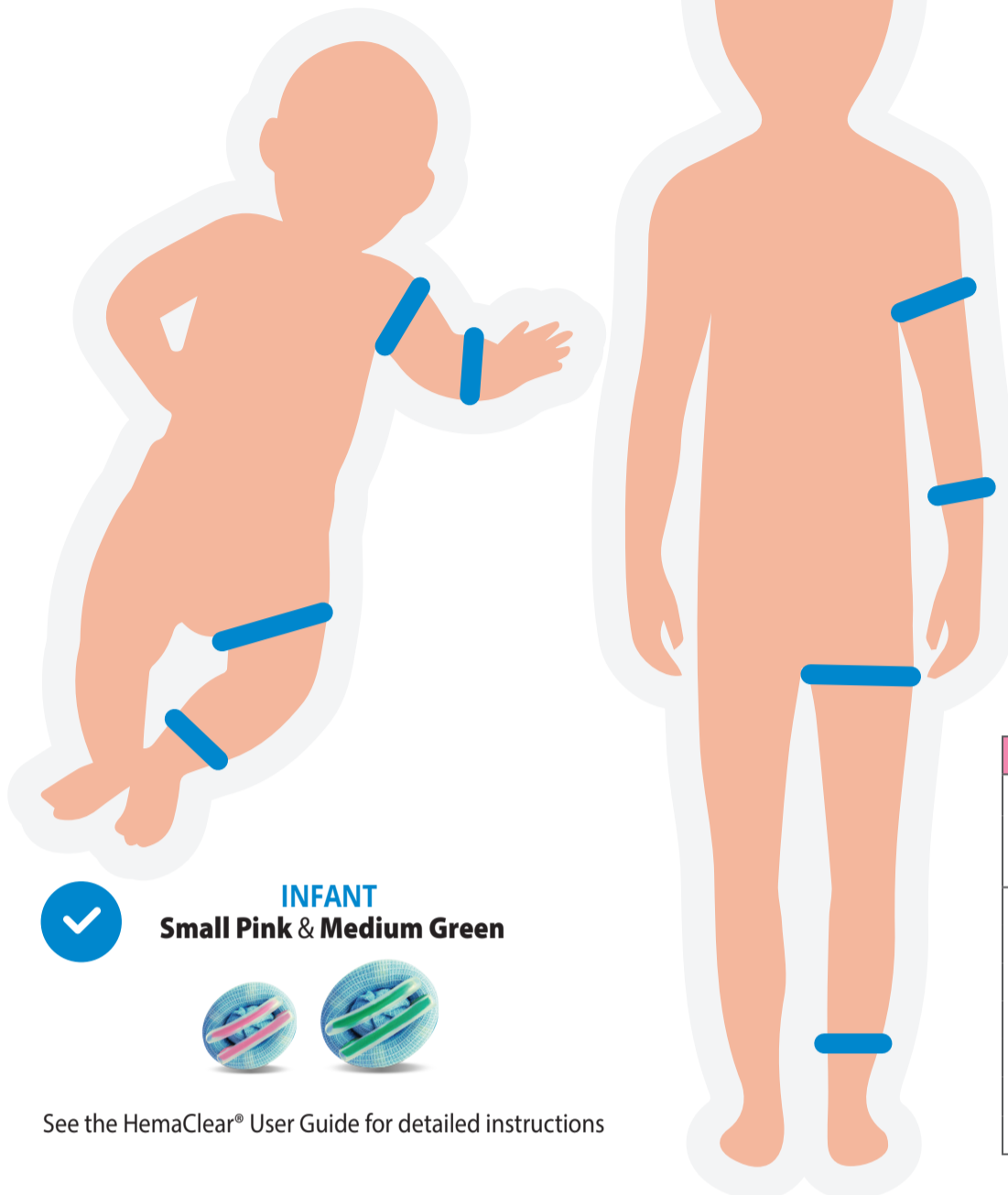


**INFANT**  
Small Pink & Medium Green

**CHILD/ADOLESCENT**  
Medium Green & Red  
Large Blue & Orange  
Model F (Forearm)

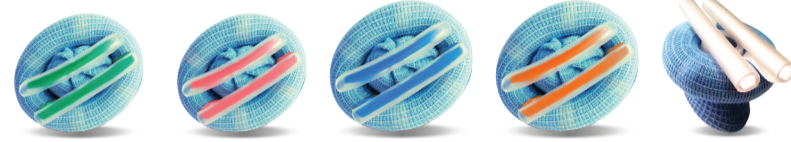

**ADOLESCENT/OBESE**  
Extra Large Black & White

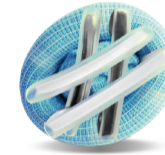

See the HemaClear® User Guide for detailed instructions

HemaClear® SMALL PINK <130 mm Hg PRH-028-PI-01A		Distance from Fingers / Toes (cm)							
		10-15	16-20	21-25	26-30	31-35	36-40	41-45	46-50
LIMB CIRCUMFERENCE (cm)	14-16	181	173	168	162	157	151	147	142
	17-18	195	189	184	178	173	169	164	160
	19-20	207	202	197	192	188	184	177	174
	21-22	217	212	207	204	198	194	187	183
	23-24	225	220	216	211	207	202	197	193
	25-26	233	229	224	219	217	209	204	201
	27-28	241	235	230	226	221	216	213	209

HemaClear® MEDIUM RED <160 mm Hg PRH-040-RE-01A		Distance from Fingers / Toes (cm)					
		20-30	31-40	41-50	51-60	61-70	71-80
LIMB CIRCUMFERENCE (cm)	24-26	221	218	212	203	194	179
	27-28	230	225	218	209	198	185
	29-30	238	235	227	217	208	191
	31-32	258	252	242	232	221	204
	33-34	265	263	257	246	235	217
	35-36	300	289	277	261	248	229
	37-38	NPD	332	301	283	262	242
	39-40	NPD	361	327	322	305	254

HemaClear® MEDIUM GREEN <130 mm Hg PRH-040-GR-01A		Distance from Fingers / Toes (cm)					
		20-30	31-40	41-50	51-60	61-70	71-80
LIMB CIRCUMFERENCE (cm)	24-26	217	213	203	195	179	162
	27-28	220	217	209	202	187	169
	29-30	232	228	218	210	193	175
	31-32	249	244	233	224	207	188
	33-34	260	253	248	239	221	200
	35-36	287	279	268	253	235	213
	37-38	NPD	324	292	273	238	226
	39-40	NPD	358	318	293	254	239

HemaClear® LARGE ORANGE <160 mm Hg PRH-060-OR-01A		Distance from Fingers / Toes (cm)						
		40-50	51-60	61-70	71-80	81-90	91-100	101-110
LIMB CIRCUMFERENCE (cm)	30-32	281	276	264	257	250	236	229
	33-36	341	332	320	311	306	285	281
	37-40	365	355	344	337	326	307	302
	41-44	NPD	423	390	363	346	328	324
	45-48	NPD	NPD	413	372	352	335	324
	49-52	NPD	NPD	NPD	387	357	341	324
	53-55	NPD	NPD	NPD	415	365	349	346

HemaClear® LARGE BLUE <130 mm Hg PRH-060-BL-01A		Distance from Fingers / Toes (cm)					
		40-50	51-60	61-70	71-80	81-90	91-95
LIMB CIRCUMFERENCE (cm)	30-32	264	257	250	236	229	216
	33-36	326	311	306	285	281	261
	37-40	363	342	326	307	302	293
	41-44	NPD	371	348	328	324	324
	45-48	NPD	NPD	357	335	324	326
	49-52	NPD	NPD	NPD	341	329	328
	53-55	NPD	NPD	NPD	347	341	335

HemaClear® EXTRA LARGE BLACK & WHITE <160 mm Hg PRH-090-BW-01A		Distance from Toes (cm)					
		60-70	71-80	81-90	91-100	101-110	111-120
LIMB CIRCUMFERENCE (cm)	50-54	300	296	292	289	285	281
	55-60	315	311	307	304	300	296
	61-66	327	323	320	316	313	310
	67-72	337	333	330	326	323	320
	73-78	345	342	338	335	331	328
	79-85	352	349	346	342	339	336

HemaClear® MODEL F™ (Forearm) <160 mm Hg PRH-035-FA-01A		Distance from Fingers (cm)		
		25-30	31-35	36-40
LIMB CIRCUMFERENCE (cm)	14-18	204	197	192
	19-22	236	231	227
	23-26	257	254	247
	27-30	276	270	264
	31-34	295	286	281