Sinus Tarsi Implant Instrument Set

Doc.: HYP Tray Insert Rev. 15
EN-English

Sinus Tarsi Implant Instrument Set Contents: Six (6) Trial Sizers (Sizer 05, Sizer 06, Sizer 07, Sizer 08, Sizer 09, Sizer 10) / *3 Guide Wires/1 Driver

Caution:

- Instrument Set is provided **NOT STERILE - MUST BE CLEANED AND STERILIZED PRIOR TO USE, SHORTLY AFTER USE & BETWEEN USES.**
- Federal law (US) restricts this device to sale, distribution and use by or on the order of a physician.
- Guide Wires must be inspected prior to sterilization and prior to each surgery. Guide Wires should be free of burrs, bends, bows or nicks as these conditions may prevent implant from sliding off freely. Defective Guide Wires should be discarded and replaced.**

Instructions for Cleaning & Sterilization of the Instrument Set

Clean Instrument Set:

- Rinse under running tap water for one (1) minute
- Immerse in Enzol (or equivalent enzymatic cleaner) solution, prepared in accordance with manufacturer's instructions using sterile Deionized (DI) water
- Scrub interior and exterior surfaces with an instrument brush two (2) times
- Rinse under running tap water for one (1) minute
- Repeat cleaning steps above one more time
- Dry thoroughly
- Place in a clean instrument tray

Sterilize Instrument Set

- Autoclave Sterilization - Prevacuum Cycle
- Double wrap instruments and tray using non woven CSR sterilization wrap (only FDA cleared wraps should be used)
- Method: Steam
- Cycle: Prevac
- Temperature: 270 degrees F (132 degrees C)
- Exposure Time: 4 minutes
- Drying Time: 30 minutes

For Immediate-Use Instrument Set may be sterilized as follows:

- (Flash) Autoclave Sterilization - Prevacuum Cycle
- Unwrapped instruments and tray
- Method: Steam
- Cycle: Prevac
- Temperature: 270 degrees F (132 degrees C)
- Exposure Time: 3 minutes
- Drying Time: Not Applicable

Emergo Europe
Molenstraat 15
2513 BH, The Hague
The Netherlands
Tel: (+31) 70 345 8570
Fax: (+31) 70 346 7299

Manufacturer:
GraMedica
16137 Leone Drive
Macomb, MI 48042
USA
Phone: +1- 586-677-9600

Emergo Europe
Molenstraat 15
2513 BH, The Hague
The Netherlands
Tel: (+31) 70 345 8570
Fax: (+31) 70 346 7299

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